


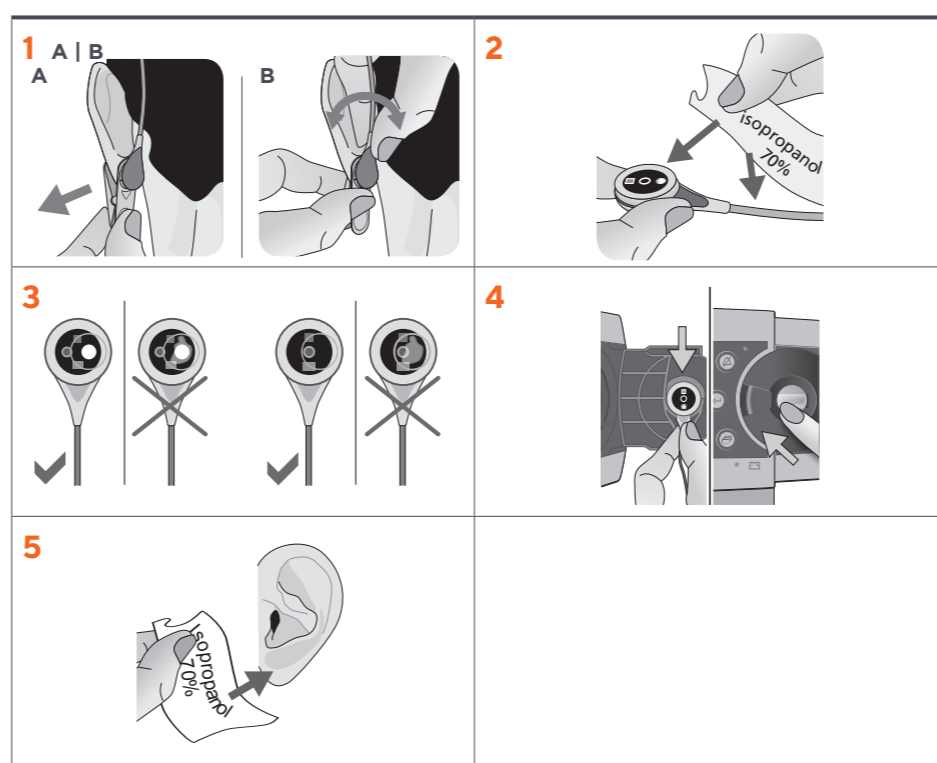
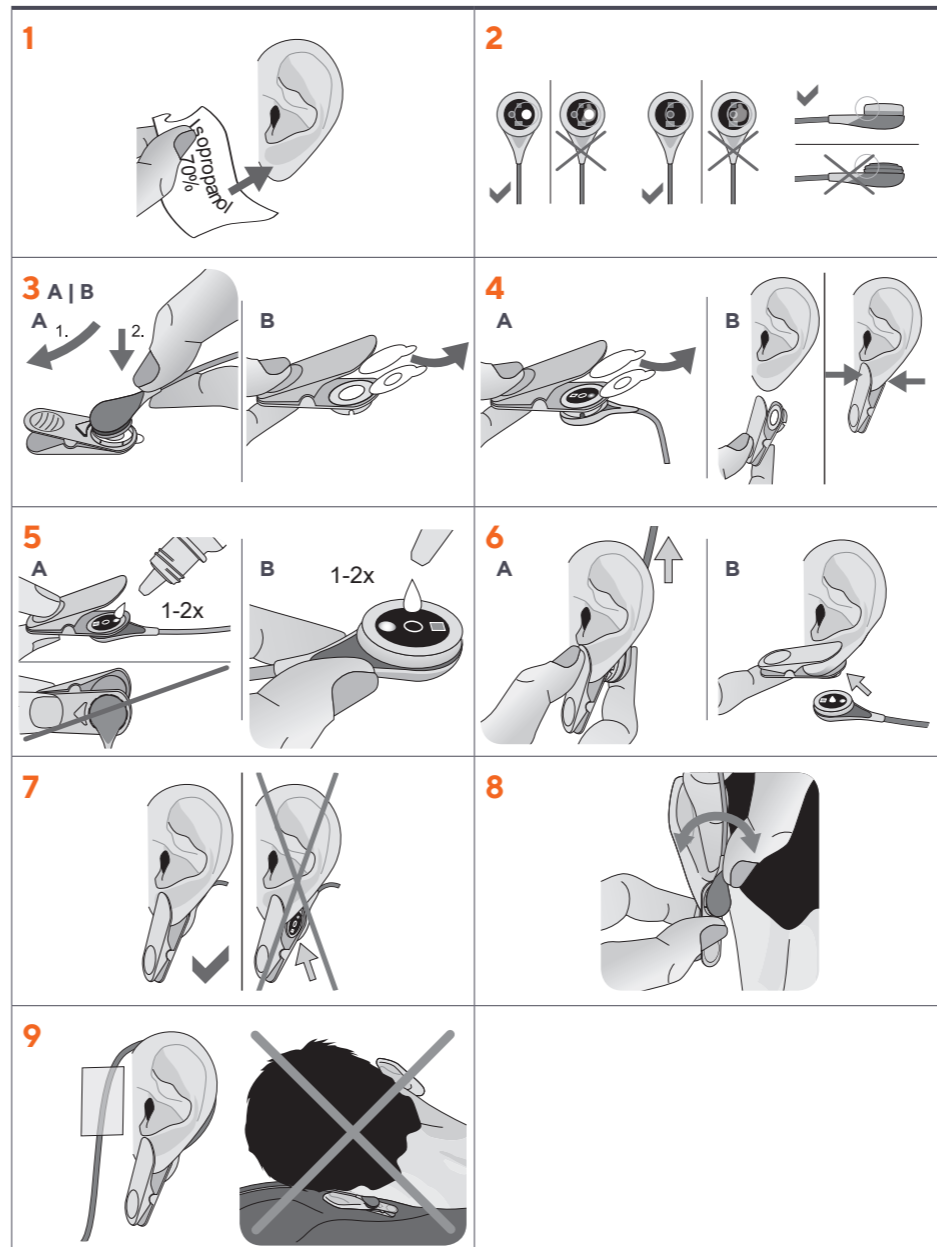
<p><b>VS-A/P/N *</b> PCO<sub>2</sub>, SpO<sub>2</sub>/PR</p> 	<p><b>V-Sign™ Sensor 2</b> en V-Sign™ Sensor 2 de Sensor V-Sign™ 2 es Capteur V-Sign™ 2 fr pt Sensor V-Sign™ 2 it Sensore V-Sign™ 2 zh OxiVent™ 传感器 ja V-Sign™ センサ ar مستشعر V-Sign™ 2 bg Сензор V-Sign™ 2 cs Senzor V-Sign™ 2 da Adbmåttæ V-Sign™ 2 el V-Sign™-i andur 2 fi V-Sign™-sensori 2 hu Jutiteles V-Sign™ 2 lv Sensors V-Sign™ 2 nl V-Sign™ Sensor 2 no V-Sign™-sensor 2 pl Czujnik 2 V-Sign™ ro Senzor V-Sign™ 2 ru Датчик 2 V-Sign™ sh V-Sign™ sensor 2 sv V-Sign™ sensor 2</p>	<p><b>IPX7</b></p>  <p>+15°C  +26°C</p> 
<p><b>OV-A/P/N *</b> PCO<sub>2</sub>, PO<sub>2</sub>, SpO<sub>2</sub>/PR</p> 	<p><b>OxiVent™ Sensor</b> en OxiVent™ Sensor de Sensor OxiVent™ es Capteur OxiVent™ fr pt Sensor OxiVent™ it Sensore OxiVent™ zh OxiVent™ 传感器 ja OxiVent™ センサ ar مستشعر OxiVent™ bg Сензор OxiVent™ cs Senzor OxiVent™ da OxiVent™-sensor el Αδσμήτης OxiVent™ et OxiVent™-i andur fi OxiVent™-sensori hu OxiVent™-érzékelő it Jutiteles OxiVent™ lv Sensors OxiVent™ nl OxiVent™ Sensor no OxiVent™-sensor pl Czujnik OxiVent™ ro Senzor OxiVent™ ru Датчик OxiVent™ sh OxiVent™ sensor sv OxiVent™ sensor</p>	<p><b>IPX7</b></p>  <p>+15°C  +26°C</p> 
<p><b>EC-MI *</b></p> 	<p><b>Ear Clip for mature, intact skin</b> en Ohrclip für normale, gesunde Haut de Clip de oreja para piel madura e intacta fr Pince d'oreille pour les peaux intactes pt Clip de Orelha para pele madura e intacta it Clip Auricolare per pelli mature e intatte zh 耳夹 適合正常、健康的皮肤 ja 耳夾 クリップ 無傷な肌用 ar مك الحان للثة السليمة المتقدمة في السن bg Клипс за ухо за зряла, здрава кожа cl Klip na ucho pro zralou, neposkrozenou pokožku da Øreclip til moden, ubeskadiget hud el Κλιπ οτιού για ώριμο, άβητο δέρμα et Kõrvaklambel normaalise, terve nahaj jaoks fi Korvaklipsi ehjälle iholle hu Füleklipetű éretti, ép bőrre it Ausies spautukas normaliai, sveikai odai lv Ausis klipsis nobriedušai ādai bez bojājumiem nl Oorclip voor volgreoiden, intacte huid no Øreklipp til moden, intakt hud pl Klips nauszny do skóry dojrzałej, nienaruszonej ro Clip de urechi pentru pielea matură și intactă ru Ушная клипса для зрелой, неповрежденной кожи sh Klips za uvo za zrelo, neostetenu kožu sv Öronklamma för mogen, intakt hud</p>	<p>+10°C  +30°C</p> <p>25%  60%</p> 
<p><b>GEL-SD *</b></p> 	<p><b>Contact Gel</b> en Kontaktgel de Kontaktgel es Gel de contacto fr Gel de contact pt Gel de contato it Gel di Contatto zh 接觸凝膠 ja コンタクトジェル ar جل موصول bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλη επαφής et Kontaktgeel fi Kontaktgeeli hu Kontaktgel it Kontaktinis gelis lv Kontaktželeļa nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktini gel sv Kontaktgel</p>	<p><b>Composition</b> en Zusammensetzung de Componentes es Constituents fr pt Componentes it Componenti zh 组件 ja コンポーネント ar المكونات bg Компоненти cs Součásti da Komponenter el Συστατικά στοιχεία et Osad fi Osat hu Komponensek it Componenti lv Sastāvdaļas nl Componenten no Komponenter pl Składniki ro Componente ru Компоненты sh Komponente sv Komponenter</p>
<p><b>GEL-04 *</b></p> 	<p>(~90%) H<sub>2</sub>O, 1,2-Propanediol (C<sub>3</sub>H<sub>8</sub>O<sub>3</sub>), Glycerol (C<sub>3</sub>H<sub>8</sub>O<sub>3</sub>)</p>	<p>+10°C  +30°C</p> 

\* The components listed above do not necessarily correspond to the scope of delivery.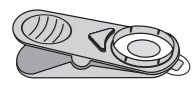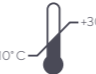

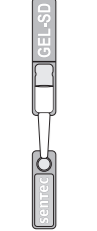en Please consult the Instruction Manual of the Sentec Digital Monitoring System for complete descriptions of warnings/cautions, recommendations of temperature setting and site time combinations, and additional information.

**Contraindications**  
The use of the Ear Clip is contraindicated in case of:  
- patients whose earlobes are too small to ensure adequate sensor application (e.g. neonates).

**Warnings/Cautions**  
The use of temperatures higher than 41°C requires special attention to patients with susceptible skin, e.g. neonates, geriatric patients, burn victims, patients with skin diseases. Ensure that all parts are applied correctly and physically intact and that the sensor is not damaged by sharp-edged objects or fingernails. Incorrect application and damage can cause incorrect measurements. Always apply sensor and Contact Gel on intact and closed skin sites. Pierced earlobes may result in incorrect tPCO<sub>2</sub> and/or tPCO<sub>2</sub> measurements. Do not apply on patients who exhibit allergic reactions to these disposables. Do not use the

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**Sentec AG**